

Clear end in sight

NEW Single-Use Spackman Cannula

SELF CLAMPING
GREAT SEAL
NO LITTLE SCREWS

TRADITIONAL SPACKMAN CANNULA

Closed end challenges
decontamination

TGA flagged as an
infection control issue



FIBERTECH
MEDICAL AUSTRALIA PTY LTD

PHONE 1800 241 881



Disposable Spackman Cannula

A cost effective solution to infection control issues with reusable instruments.

The closed end and fine holes of a reusable Spackman cannula are described by CSSD managers as "impossible to clean effectively". Debris loosened by autoclaving can be carried into the uterus and fallopian tubes during injection of dyes, potentially causing serious uterine infections. In Australia, TGA have also flagged this reusable instrument as an infection control issue – we have the solution:

The best way to ensure 100% Infection Control is a disposable version of the re-usable instrument. Medica 2004 saw SUSL (UK) launch their single use Spackman Cannula that is the same size and shape as the traditional proven design, but has an improved clamp with an easy self-locking action and no small screw that can be lost. Fibertech Medical Australia was present for this launch and has secured the distribution rights to Australasia for this product.

Response has been enthusiastic from surgeons throughout the World describing it as "easy to insert and to use".

SUSL (UK) have taken the reusable instrument and matched the geometry for equivalent uterine manipulation capability, but improved the clamp action. As the traditional instrument it has a well rounded distal end for easy placement through the endocervical canal and transverse holes at the distal end for even dye distribution. The clamp incorporates reinforced hooks for tenaculum traction and the handle accepts a standard luer for injection of dye into the uterine cavity.

The main features and benefits of the S029 are:

- Single use instrument – reduced risk
- No clamp screw to drop or lose
- Good seal on cervix
- Latex free and DEHP free
- Quick and easy to insert and remove

FOR MORE INFORMATION OR TO ARRANGE A TRIAL

Please call Fibertech on 1800 241 881 or fax the following details to 03 9545 5587

HOSPITAL _____

CONTACT NAME _____

PHONE _____ BEST TIME TO CALL _____